

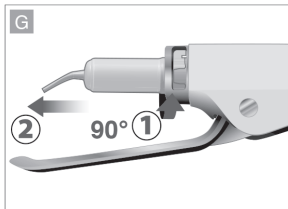
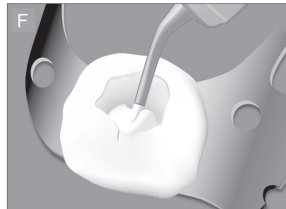
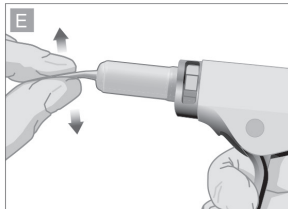
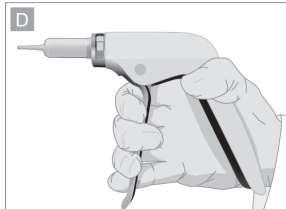
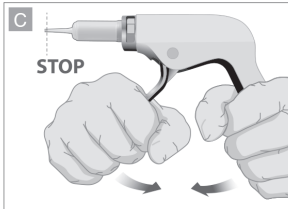
# Biodentine™ Gun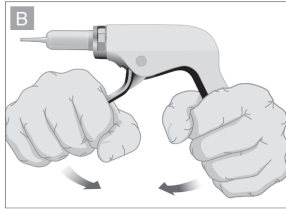

fr	Dispositif pour extrusion de substitut dentinaire	1
en	Device for extrusion of dentine substitute	4
es	Dispositivo para la extrusión de sustituto de dentina	7
de	Vorrichtung zur Extrusion von Dentinersatz	10
it	Dispositivo per l'estrusione del sostituto della dentina	13
pt	Dispositivo para extrusão de substituto dentinário	16
nl	Apparaat voor extrusie van dentinesubstituut	19
pl	Przyrząd do wyciskania substytutu zębiny	22
el	Ιατροτεχνολογικό προϊόν για την εξαγωγή του υποκατάστατου οδοντίνης	25
bg	Устройство за екструзия на заместител на дентин	28
cs	Zařízení pro vytlačení dentinové náhrady	31
da	Instrument til udtrækning af dentinerstatninger	34
et	Seade dentiini asendaja ekstrusiooniks	37
hu	A dentinpótló anyag extrudálására szolgáló eszköz	40
lt	Prietaisas išstumti dentino pakaitalą	43
lv	Ierīce dentīna aizstājēja ekstrūzijai	46
ro	Dispozitiv pentru scoaterea prin presare a substitutului dentinar	49
sk	Zariadenie na vytlačanie dentinovej náhrady	52
sl	Naprava za iztiskanje nadomestka dentina	55
sv	Anordning för extrudering av dentinsubstitut	58
no	Enhet for ekstrudering av dentinerstatning	61

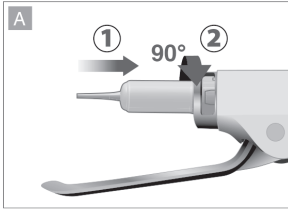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Biodentine™ Gun is a reusable device used in compatibility with Biodentine™ XP.

## **PRESENTATION**

Box containing one Biodentine™ Gun

## **INTENDED USE**

Biodentine™ Gun is intended to be used to extrude dentine substitute from Biodentine™ XP cartridges.

## **TARGET GROUP(S)**

### **Intended users**

Dental healthcare professional only

## **PERFORMANCE CHARACTERISTICS**

### **Technical performances / specifications**

Extrusion of a dentine substitute from a cartridge

## **INDICATIONS FOR USE**

Biodentine™ Gun is used to extrude dentine substitute from Biodentine™ XP cartridges for dental healthcare.

Biodentine™ Gun is reusable and has the ability to undergo 325 reprocessing cycles.

## **CONTRA INDICATION**

There are no contra indications.

## **EXPECTED LIFETIME**

Biodentine™ Gun can be used up to 325 reprocessing cycles

## **INSTRUCTIONS FOR USE:**

### **Refer to precautions for use 1 and 2 and warning 1**

#### **1. Biodentine XP cartridge preparation**

- 1.1. Wear the appropriate personal protective equipment
- 1.2. Refer to Biodentine™ XP instructions for use.

#### **2. Insertion of the Biodentine™ XP cartridge in Biodentine™ Gun**

- 2.1. Raise the Biodentine™ Gun lever until it stops in horizontal position
- 2.2. Align the cartridge with Biodentine™ Gun (See picture A, step 1)
- 2.3. Insert the cartridge into Biodentine™ Gun and lock the cartridge making a clockwise 90° turn (See picture A, step 2)

#### **3. Priming**

Hold Biodentine™ Gun, the lever with one hand, the handle with the other hand and push progressively down the lever. (Refer to precaution for use 5, warning 9 and see picture B) Stop when the dentine substitute appears in the nozzle (See picture C)

#### **4. Extrusion**

- 4.1. Hold Biodentine™ Gun with one hand (See picture D)
- 4.2. The cartridge nozzle can be bent manually using adequate strength (due to thickness of the nozzle), to facilitate the placement of the dentine substitute (Refer to warning 11 and see picture E)
- 4.3. Push progressively the lever down to allow the dentine substitute flow out of the cartridge. A homogeneous and creamy paste is obtained

#### **5. Application**

The dentine substitute can be directly applied into the affected tooth cavity with Biodentine™ Gun. Refer to Biodentine™ XP instructions for use for other application methods

#### **6. Cartridge removal**

- 6.1. Release the lever and unlock the cartridge from Biodentine™ Gun making an anti-clockwise 90° turn (See picture F step 1) then remove the cartridge (See picture F step 2)
- 6.2. Dispose the cartridge in an appropriate bin. Refer to Biodentine™ XP instructions for use

#### **7. Processing**

Below are the recommended processing methods. Refer to warning 2 and 3

## **Manual**

### Equipment required:

Not applicable

### Method:

Rinse and soak Biodentine™ Gun in a non-glutaraldehyd disinfectant

Exposure time: at least 12min

Brush manually

Rinse under water

Dry during 30 minutes

Refer to precaution for use 7

## **Automated**

### Equipment required:

Ultrasonic cleaning machine

Enzyme detergent

Brush

### Method:

Maximum exposure temperature: 40°C (104°F)

Maximum exposure time: 20 minutes

Brush manually

Rinse under water

Dry during 30 minutes

## **Sterilization**

### Equipment required:

Type B vacuum autoclave

Packaging pouch compatible with type B vacuum autoclave. Refer to warning 6

### Method:

Wrap Biodentine™ Gun in an appropriate pouch for sterilization

Maximum exposure temperature: 135°C (275°F)

Maximum exposure time: 18 minutes

The instructions provided above have been validated by the manufacturer as being capable of preparing a device for reuse. It is the processor's responsibility to ensure that the processing achieved the desired result whatever the method, equipment and staff involved. This requires verification and/or validation and routine monitoring of the process.

## **Inspection and maintenance**

Visually inspect all surfaces. If any soiling is still visible, repeat processing step.

## **WARNINGS AND PRECAUTIONS FOR USE**

### **Precautions for use**

1. Wear the appropriate personal protective equipment while manipulating the device.
2. All the instructions must be carried out within the working area, far from the patient.
3. Refer to Biodentine™ XP instructions for use for cartridge preparation.
4. Use Biodentine™ Gun only with Biodentine™ XP cartridges otherwise, technical performances are not guaranteed.
5. Do not prime a cartridge, which is not locked at 90° in the Biodentine™ Gun.
6. Be cautious while handling and using Biodentine™ Gun so as not to inadvertently cause harm.
7. Oxidizing disinfectant solutions are not suitable because they alter the performances of the Biodentine™ Gun.

### **Warnings**

1. Process the Biodentine™ Gun before first use because it is delivered non sterilized.
2. Do not process for more than 325 cycles. Otherwise, technical performances are not guaranteed.
3. Do not use Biodentine™ Gun after 325 cycles. Otherwise, technical performances are not guaranteed.
4. Do not use a damaged Biodentine™ Gun; it alters the performances of the device.
5. To avoid contamination between patients, ensure to always use a processed Biodentine™ Gun.
6. Use one pouch per Biodentine™ Gun, otherwise the sterilization is not guaranteed

7. It is strongly recommended to use a rubber dam to protect patient.
8. If the Biodentine™ Gun is sterilized after use, keep it in the sterilization pouch until next use.
9. During priming step, if the lever is too strongly pushed, it may result in a dentine substitute's release going out of the cartridge.
10. Biodentine™ Gun is only intended to extrude dentine substitute from Biodentine™ XP cartridges.
11. Do not bend the nozzle more than 4 times to avoid any breakage.

### **SIDE-EFFECTS**

There are no known side effects.

Please report any serious incident that occurred in relation to the device to Septodont ([vigilance@septodont.com](mailto:vigilance@septodont.com)) and to the local competent authority for medical devices.

### **STORAGE**

No specific condition.

### **DISPOSAL**

Dispose Biodentine™ Gun in a regular bin.

### **SYMBOL**

<b>Symbol</b>	<b>Identification</b>
	Manufacturer
	Distributor
	Unique device identifier
	Catalogue number
	Batch code
<b>Rx only</b>	For Professional Use Only
	CE marking of conformity
	Date of manufacture
	Consult the instruction for use
	Sponsor

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**SEPTODONT Inc**